




# I-NOX LAB

## Workflow

### 1 - Case reception

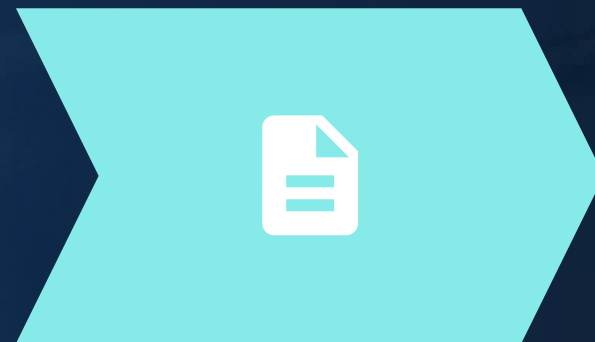


Receiving all the data required for the case

### 2-Planning & Designing




Specialists dentist team planning your case

### 3 - Review session

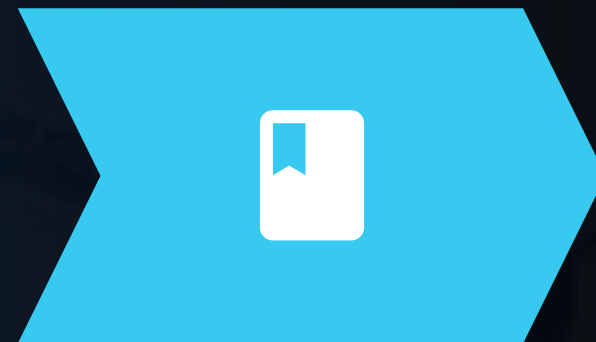


Review session with our dentists

### 4 - Production

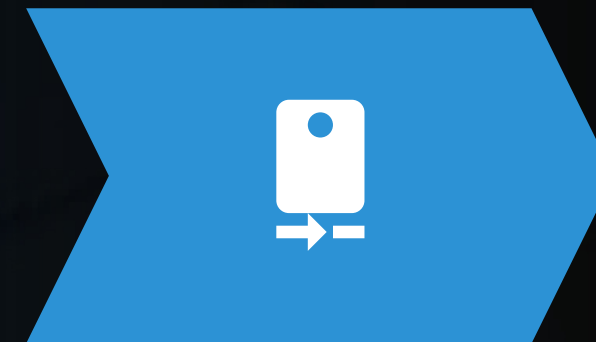

### 5 - Delivery

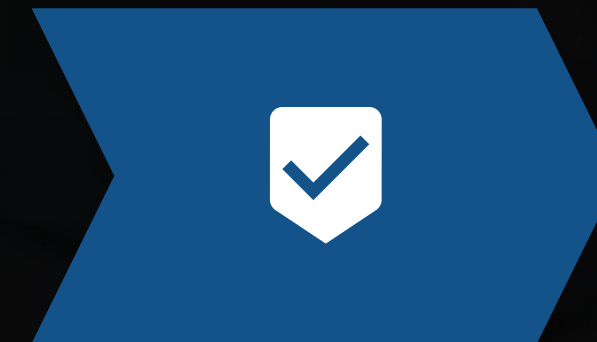

Delivery within 5 business days and chairside assistance

# Requirements for Full arch cases

**1- GOOD QUALITY CBCT OR DUAL SCAN  
PROTOCOL FOR EDENTULOUS  
PATIENTS**

## **2- STLS**

STL for the patient with and without the denture.


STL for the fitting surface of the denture.

STL for the antagonist and Bite

## **2-DSD PHOTOS**



# B-The 3 visits Final solution

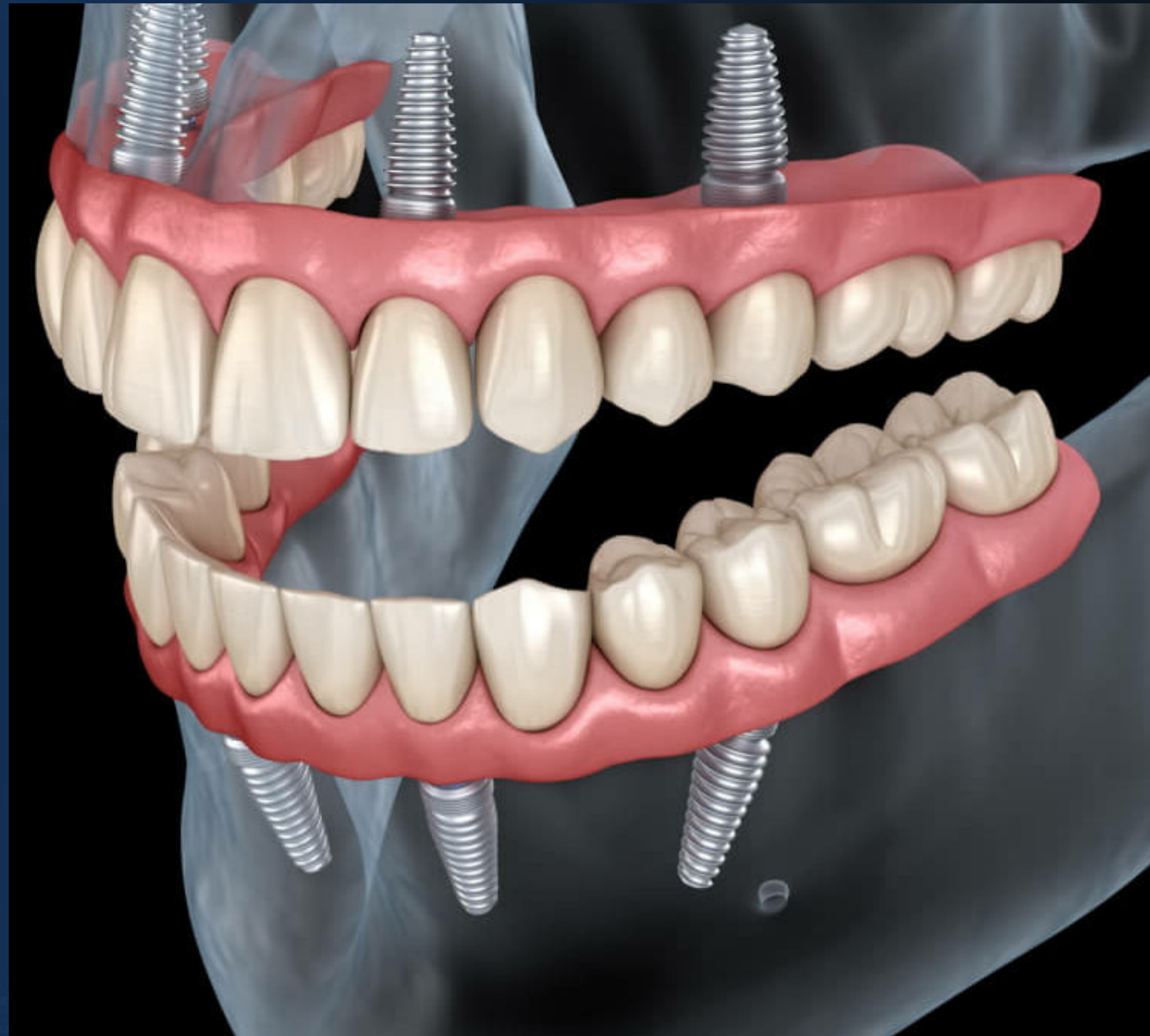

Must be done within 3 to 7 days time frame  
Primary stability must be more than 35N  
Abutment Level impression  
Data acquisition precision is very crucial

# 1st visit

## WHAT YOU WILL RECEIVE?

1-CUSTOMIZED  
SURGICAL GUIDES

2-PICKUP PMMA

## DATA TO ACQUIRE ?

- IMPLANTS POSITION  
REPLICATION
- SOFT TISSUE REPRESENTATION
- ADJUSTED BITE REGISTRATION





# Double digital scanning method

**AFTER PMMA PICKUP AND REQUIRED ADJUSTMENTS AND WHILST LEAVING THE GUIDE FIXATION PINS IN POSITION**

1-Titanium powder To be sprayed on the soft tissues to facilitate the intraoral scanning and reduce reflection

A digital impression to be taken with the fixed PMMA in position (covering as much surrounding soft tissue and fixation pins)

-Antagonist and bite STLs as well to be taken

The prosthesis to be removed, and a second digital impression to be taken with scan bodies tightened onto the multi-unit abutments with the same fixation pins in place as well (to serve as fiducial landmarks to merge both STLs)

Index cast to be taken with the prosthesis in position


# Conventional method

**AFTER PMMA PICKUP AND REQUIRED ADJUSTMENTS HAVE BEEN MADE.**

Multi-unit abutment level Splinted open tray PVS impression to be taken

Bite registration

Antagonist impression

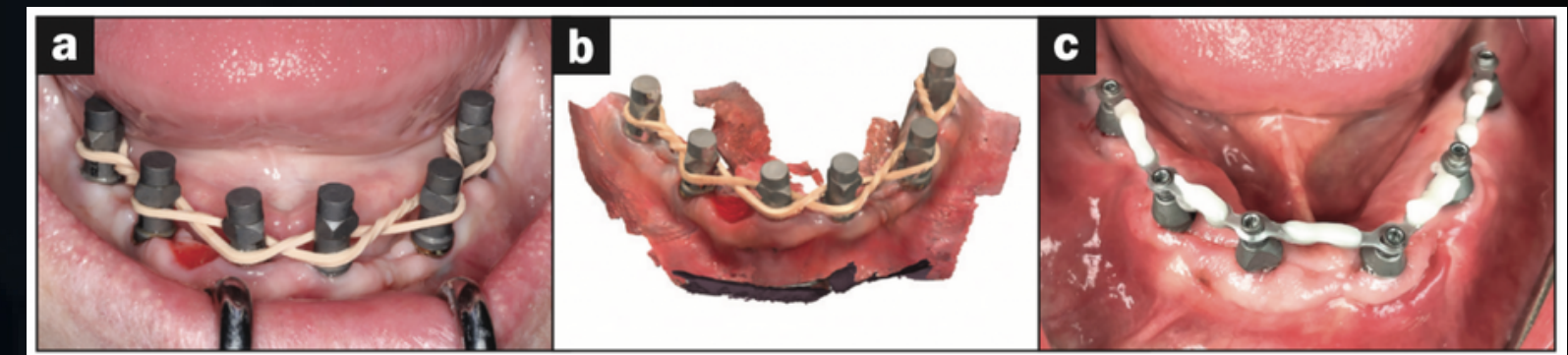

**\*Please note that an extra visit will be required for verification Jig in case of Conventional PVS impression**



# 2nd visit

**WHAT YOU WILL RECEIVE?**

**TRY IN FOR THE FINAL  
RESTORATION**

**VERIFICATION JIG (FOR  
CONVENTIONAL METHOD)**

**DIGITAL**



# **3RD VISIT (FINAL VISIT)**

**WHAT YOU WILL RECEIVE?**


**SCREW RETAINED FINAL  
FULL ARCH NANOKSA G-  
PLUS RESTORATION**









 30 Ngould Ste R Sheridan, WY82801  USA

 Egypt | For North Africa  
**Cairo Office**

 UAE | For Gulf Region  
**Dubai Office**

 Turkey | For Asia Region  
**Istanbul Office**

 [WWW.i-nox.us](http://WWW.i-nox.us)

 | [info@i-nox.us](mailto:info@i-nox.us)  
[Info@i-noxmena.net](mailto:Info@i-noxmena.net)